

| STRETCHER INSPECTION AND MAINTENANCE | | |
|--|-------------|------------|
| Unit # _____ Date _____ | | |
| | YES | NO |
| Is cot free of excessive wear? | | |
| Are all screws, nuts, bolts, rivots and roll pins securely in place? | | |
| Do all moving parts operate smoothly and properly? | | |
| Does the cot lock properly in each position? | | |
| Does the cot load and unload properly? | | |
| Does safety hook properly engage during unloading? | | |
| Does the cot roll smoothly? | | |
| Do the wheels have some tread? | | |
| Do the wheel locks engage properly? | | |
| Are the restraints properly installed? | | |
| Is restraint webbing in good condition, no cuts or frays? | | |
| Are restraint buckles free of damage and operate properly? | | |
| Do installed accessories operate properly without interfering with operations? | | |
| Did you properly lubricate the strecher? | | |
| | | |
| Stretcher deficiencies or problems | | |
| Issue | Reported to | Date fixed |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| Stretcher Checked by: _____ | | |